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## ASSEMBLING AND ADJUSTMENT INSTRUCTIONS FOR 1P110 HYDRAULIC KNEE

### USE :

- For active patients or patients with low activity
- Up to 100 kg
- With a good muscular control
- Who expect a smooth and comfortable walk.

### CONTENTS :

- 1 hydraulic knee 1P110
- 1 instruction sheet.

### CHARACTERISTICS :

- Control of the swing phase : extension return and flexion are independent
- Adjustment of damping during extension
- Weight : about 870 g.
- Flexion : up to 145°

### MATERIALS :

Titanium, aluminum alloy of high quality, ...

## ASSEMBLING

### **Upper link :**

- 1K03 or 1K30 : Plate with 4 holes and 36 mm centre distance  
1K160 or 1K163 : Pyramid connector  
1K40 : Connector for lamination  
1K20 : Extension tube for knee

*Note : The assembly instructions for the above components must be carefully respected.*

**B :** Theoretical rotation axis


### **Lower link**

1G01 : 30 mm Ø tube

**D :** Screw tightening torque : 11 Nm

### **Finishing**


- 1G13 (complete or partial) : single-block cosmetic cover  
1G28 and 1P11090 : 2-parts cosmetic cover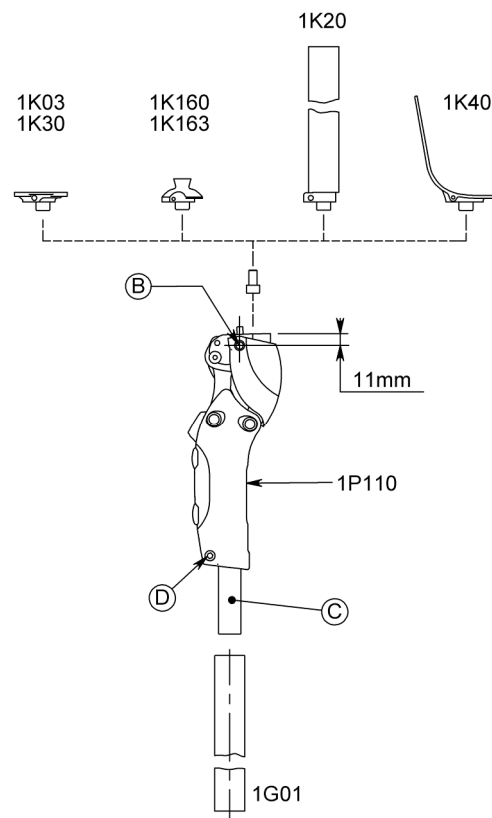

### **IMPORTANT INFORMATION**

- Mount the 1G01 tube on the stop of the bore bottom.
- Do not exceed the recommended value for the tightening torque of screw **D** because this could damage the tube.

With shorter leg segments or in order to use bulkier feet, you can replace **C** adjustment cylinder by our shorter **1P11067** component.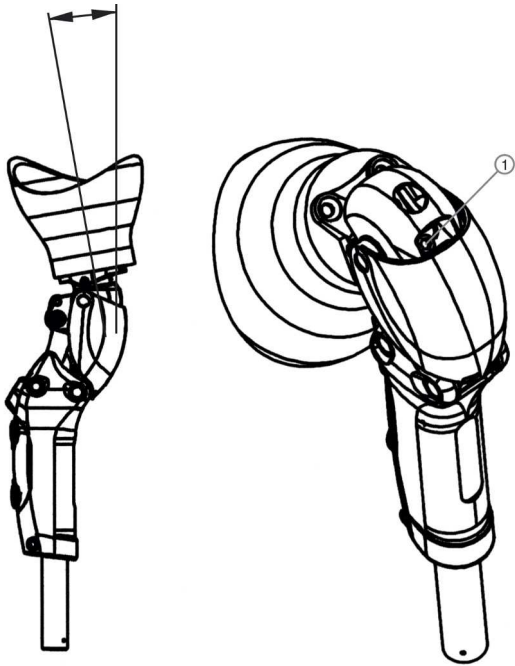

## STABILITY ADJUSTMENT


Adjustment of the extension stop :

- Bent the knee to access the adjustment screw.
- Screw M6 screw (1) out, which increases knee flexion.  
Stability decreases, but the start of knee flexion during walking is smoother.


*Note :*  
*Factory adjustment corresponds to maximal hyper-extension.*

## RECOMMENDED ALIGNMENTS

**Alignment for foot like SACH foot**  
(for other feet, please respect the manufacturer instructions)

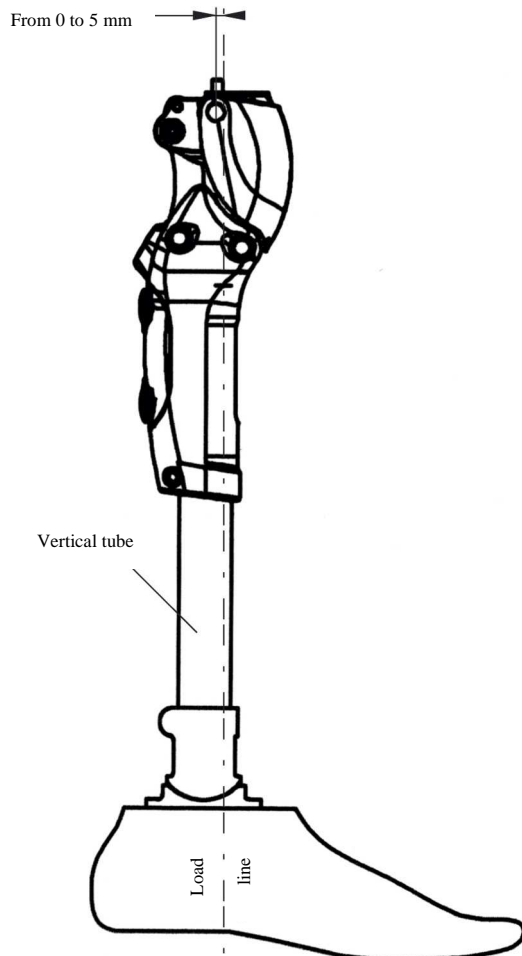**Alignment for single axis foot**
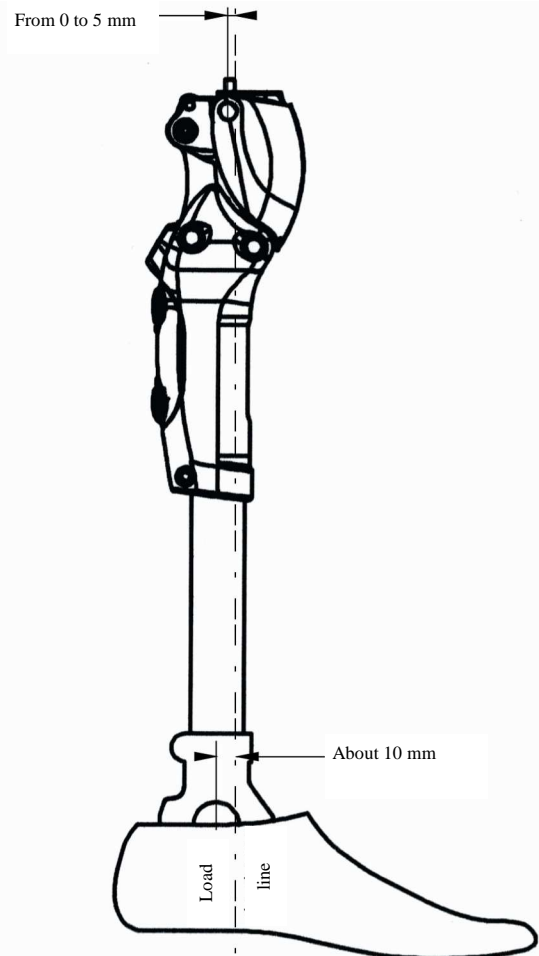
## WALKING PACE ADJUSTMENT

The HYDEAL knee comes with factory adjustments for a «standard» walking pace (*buttons A and B in position 3, and cylinder C screwed home unless 3 turns*).


To personalize these adjustments according to the patient's normal walking pace, proceed in the following order to the adjustment of :

1) Resistance to flexion during walk : screw in (to decrease flexion angle) or out (to increase flexion angle) the lower adjustment button **B**.

2) Resistance to extension : screw in (to increase damping) or out (to decrease damping) the upper adjustment button **A**.

If the optimal adjustment of the patient's walking pace could not be reached, use cylinder **C** to modify the power of the extension assist : screw in to increase extension assist (quicker return) or out to decrease extension assist (slower return).


**CAUTION : In any case check that the knee can reach its optimal flexion.**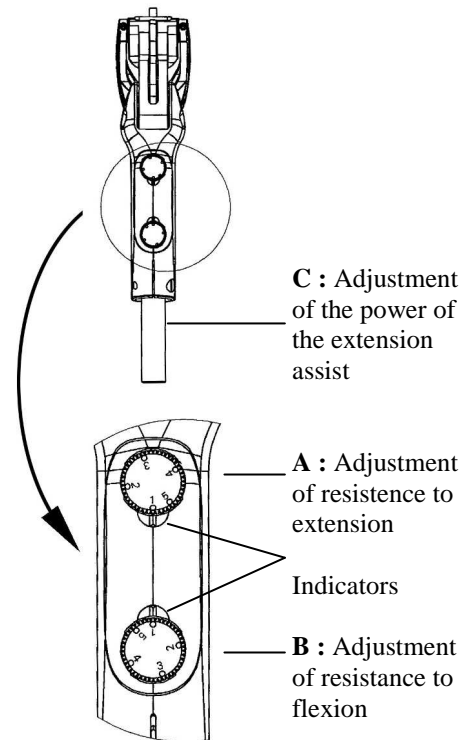

## FLEXION END STOP

When the patient is crouching or kneeling, the socket must be in contact with the flexion end stop.

Check that in these conditions there is no contact between the linkage and the plate.

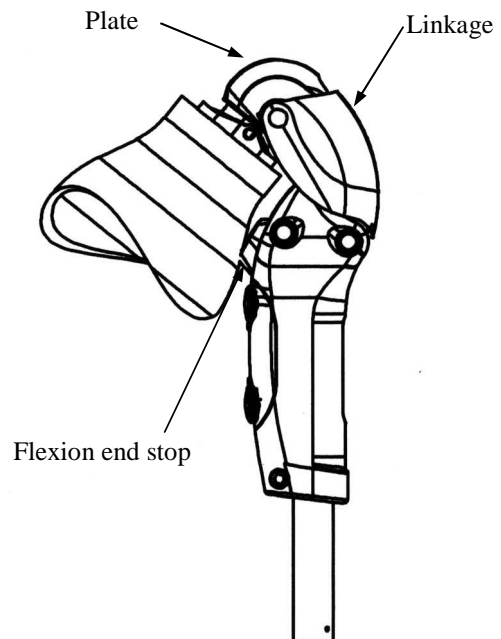## MAINTENANCE

This product requires only a yearly visit to your prosthetist to check that all the system works well.

## WARRANTY

This product is guaranteed against any manufacturing defect for one year. This warranty is not applicable to damages due to a bad assembly, an inadequate alignment, a dusty environment without adapted protection, or any inadequate use.

**CAUTION :** Use a silicon spray to eliminate noises due to cosmetic foam cover friction. Do not use talcum powder. Talcum powder reduces lubrication of mechanical parts, which may lead to function defects and risk of falling for the patient.

**The application of talcum powder will render all claims against Proteor null and void.**

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